

Issue Date:

Review Before:

***Title: SOP FOR CLEANING OF AUTOCLAVE.***

---

---

**OBJECTIVES:**

To lay down a procedure for Cleaning of Autoclave.

**SCOPE:**

This procedure is applicable for cleaning of Autoclave at CSSD, PGIMS, Rohtak.

**RESPONSIBILITY:**

Sterilisation Technician, CSSD Attendants, Bearers.

**PROCEDURE:**

1. Wipe the outer surface of autoclave with a wet sponge, clean cloth and pipe connections with clean dry cloth every day before operating autoclave.
2. Clean the chamber once in a week with detergent solution using brush and wash with tap water to remove detergent. Then wash with demineralised water. Finally wipe the chamber with clean dry cloth.
3. Sterilization Technician will ensure the cleaning procedure & completion.

	Prepared By	Checked & Approved by	Authorized By
Designation	St. technician	Asstt. Supervisor	Officer In charge
Name	Anil Kumar	Yogesh Kumar	P.K. Verma
Sign/ Date			