

Issue Date:

Review Before:

## ***Title: SOP FOR STERILIZATION OF INSTRUMENTS TREYS***

---

---

### **OBJECTIVES:**

To lay down a procedure for Recycling of Instruments Treys received from various departments after use.

### **SCOPE:**

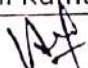
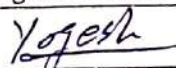
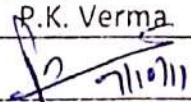
This procedure is applicable to ensure proper washing, disinfection, drying and sterilization of Instruments.

### **RESPONSIBILITY:**

Assistant Supervisor, Sterilisation Technician, CSSD Attendants.

### **PROCEDURE:**

1. Used instruments treys received from various departments through receiving window in unsterile zone of CSSD.
2. Instruments are washed in flowing water properly.
3. Disinfection of Instruments is done by 5% savlon solution.
4. Dry & clean the disinfected Instruments with clean cloth.
5. Arrange the specified instruments in specified treys.
6. Sterilization indicator/tapes are placed on the treys before loading in Sterilizer and then properly arrange the treys into Sterilizer. The steam supply has been made in Sterilizer closed through jacket to chamber and remove the air in the chamber by Saturated Steam and then maintain the temperature 121° C at 15 psi pressure for 25-30 minutes & then release/exhaust the steam from sterilizer. After that the sterilizer indicator has been checked as colour of indicator changes from green to black or grey. It shows that sterilization cycle done properly & record of sterilization cycle maintained in log book.

	Prepared By	Checked & Approved by	Authorized By
Designation	St. technician	Asstt. Supervisor	Officer In charge
Name	Anil Kumar	Yogesh Kumar	P.K. Verma
Sign/ Date			

Issue Date:

Review Before:

***Title: SOP FOR STERILIZATION OF INSTRUMENTS  
TREYS***

---

---

7. These sterilized treys are stored in sterile distribution section according to marked area of different departments and distribution is made to the departments against receipt of used treys.

+

	Prepared By	Checked & Approved by	Authorized By
Designation	St. technician	Asstt. Supervisor	Officer In charge
Name	Anil Kumar	Yogesh Kumar	P.K. Verma
Sign/ Date	